




MICROSURGICAL
TECHNOLOGY™

Spare Parts Kit
Cleaning & Sterilization
Instructions



MicroSurgical Technology Inc.
8415 154th Ave NE
Redmond, WA 98052 USA
425-861-4002
info@microsurgical.com

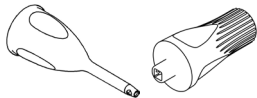

67-0052-01 Rev A

Silicone Sleeve Cleaning Instructions

Immediately after use, soak the sleeve in distilled or sterile water.

Following the Procedure:

- Prepare a solution of non-enzymatic or enzymatic neutral pH detergent in DI/distilled water following manufacturer's instructions.
- Remove the sleeve from the tip and rinse it in the solution immersing and agitating it.
- Flush 10cc of the cleaning solution through the lumen of the sleeve with a syringe.
- Rinse the surface of the sleeve with DI/distilled water.
- Flush 10cc of DI/distilled water through the sleeve.
- Dry the internal surfaces of the sleeve by flushing 10cc of air with a syringe.
- Dry the external surfaces of the sleeve with a clean, low lint cloth.
- Sterilize to your facility's standard practice or to industry standards.

MST recomenda os parâmetros de esterilização relacionados abaixo.

Não embalada por gravidade	Não embalada pré-vácuo	Embalada pré-vácuo
132°C (270°F) 15 min	132°C (270°F) 4 min	132°C (270°F) 4 min, secar 15 min