

Pouch Label

 **MICROSURGICAL TECHNOLOGY™**

MST Phaco Test Chamber

REF **IH-90339** LOT **XXXXXX**

EC REP QTY: **5**



MDSS GmbH
Schiffgraben 41
30175 Hannover, Germany

 MicroSurgical Technology Inc.
8415 154th Ave NE Redmond, WA 98052 USA
Phone 425.861.4002 email: info@microsurgical.com

50-IH90339 Rev A

Cleaning/Sterilization Label

 **MICROSURGICAL TECHNOLOGY™**

Test Chamber Cleaning & Sterilization

Cleaning This device is intended for multiple uses.

- Fill the test chamber with DI water, and then drain it. Repeat three times.
- Rinse the device in clean DI water.
- Sonicate the devices in fresh DI water for 3 minutes.
- Dry the device with clean lint-free cloth

Sterilizer type – pre-vacuum

- Configuration – individually pouched or unpouched devices
- Minimum temperature – 132° C (270° F)
- Full cycle time – 4 minutes
- Drying time – 20 minutes (pouched only)

Sterilizer type – gravity

- Configuration – individually pouched or unpouched devices
- Minimum temperature – 132° C (270°F)
- Full cycle time – 15 minutes
- Drying time – 15 minutes (pouched only)



 MicroSurgical Technology Inc.
8415 154th Ave NE Redmond, WA USA 98052
Phone 425.861.4002 email: info@microsurgical.com

51-IH90339 Rev B