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MicroSurgical Technology Malyugin Ring System® Directions for Use

Description:
The Malyugin Ring® is a continuous strand of blue polypropylene formed into a Ring with spiral scrolls at each corner. It is provided in a retaining Holder. The Malyugin Ring® System consists of the Holder with the Ring and the Inserter, which is packaged together with each Ring.

CAUTION: This product is sold in a Sterile condition and is intended for a single use. Do not attempt to re-sterilize the device.

Intended Use:

The Malyugin Ring® is used to temporarily dilate the iris of the eye, primarily in conjunction with cataract surgery. The Inserter is used to withdraw the Ring from the Holder, to introduce the Ring into the eye, and to remove the Ring from the eye.

CAUTION: The Malyugin Ring® is a delicate device handle with care. If the device appears to be damaged do not use.

Instructions

Loading the Ring into the Inserter

NOTE: Keep the Ring in the Holder until ready to use.

- Move the thumb button slowly to the rear position, pulling the Ring from the Holder into the cannula. Continue until thumb button reaches the end of the slot.
- Remove the Inserter from the Holder and discard the Holder.

Introducing the Ring into the Eye

- With viscoelastic in the Anterior Chamber inject the Ring approximately 2mm forward, then engage the distal iris margin.
- Inject while simultaneously retracting the injector. "Toeing" the injector down eases placement of the lateral scrolls.
- Lay proximal scroll on the anterior iris.
- Use an Osher-Malyugin Ring Manipulator to place the proximal scroll. Don't use a Sinskey hook.

NOTE: The Ring may appear slightly elongated when expelled from the Inserter. This will not affect the performance of the Ring.

For Smaller Pupils < 4.5 mm

- Place the tip of the cannula into an incision of 2.2mm or larger.
- Move the cannula into the anterior chamber to about the mid-pupil point. Move the slide of the Inserter forward, until about 2mm of the Ring is showing.
- Move the Ring/cannula forward until the distal Ring scroll is engaged with the distal iris.
- Inject proximal and lateral scrolls on top of the anterior iris while simultaneously retracting the injector.
- Use an Osher-Malyugin Ring Manipulator to place the proximal scroll.
- Use an Osher-Malyugin Ring Manipulator to place one lateral scroll.
- Use an Osher-Malyugin Ring Manipulator to place the other lateral scroll.

Removing the Ring from the Eye

- Use an Osher-Malyugin Ring Manipulator to push the proximal scroll into the pupillary space and then move your hand back and lay the scroll on top of the iris.
- With the thumb button in the rear position (Hook retracted), re-insert the Inserter's cannula into the eye.
- Move the slide forward to fully extend the hook. Place the cannula's platform underneath the scroll and hook above the scroll. Move the slide backward to retract the hook. Hook will close itself inside the scroll capturing the scroll and pulling it into the cannula.
- Slowly retract the thumb button, pulling the Ring into the cannula.
- Remove the cannula from the eye and discard the Inserter and the Ring.

NOTE: As an alternative to Step 1, one may use the Inserter as a pushing instrument, as described above, to disengage the proximal scroll.

CAUTION: The Malyugin Ring® is intended for single use. Do not re-use this product.

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